




To: All AMR Contra Costa Employees
From: Troy Vincent, Clinical Manager
Date: July 13th, 2020
RE: **NEW Spit Hood**

Due to availability, we have changed the make and model of the previous Spit Hood. The application and packaging are very similar. Please see the images attached and the instructions from the manufacturer.



INSTRUCTIONS FOR USE:

- 1) Open and remove the Tranzport Hood™
- 2) Place the Tranzport Hood™ over the head of the person with the mesh fabric positioned just below the eyes to allow the person to see
- 3) For the best fit, place the center elastic under the nose and over the ears. For better protection, the elastic may be placed above the nostrils. *See illustration below*
- 4) Carefully push the plastic Secure Lock Tab down toward the top of the head while holding the top of the mesh fabric. This should take the slack out of the top and help secure the Tranzport Hood™ in position. **DO NOT** push so tightly as to be uncomfortable or impair the vision of the wearer.

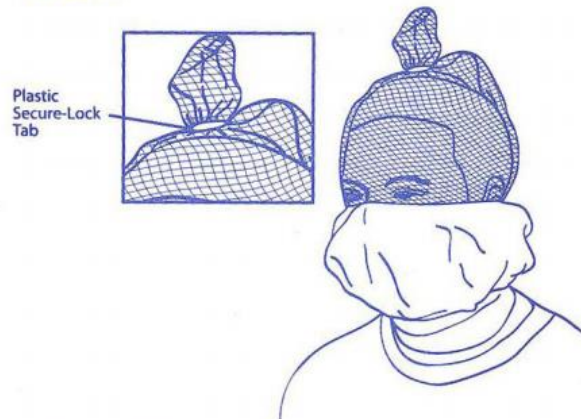

The Tranzport Hood™ is designed to be used on a single person, and should never be reused. Remove when destination is reached or the prisoner is to be left unattended, or as advised by a superior and safely discard.

Thank you for your time and attention to this matter.

CC: Troy Vincent, Clinical Manager